

Coolbrace™ Cervical Orthosis

The Coolbrace™ cervical orthosis is engineered to provide patients with the required degree of stabilization without compromising comfort. With features that include cotton-lined, breathable foam padding and forced-air technology for ventilation, the Coolbrace orthosis is designed to maximize patient comfort and compliance.



Coolbrace™ Features and Benefits

- Forced-air technology with multiple speed and ventilation settings provides quiet comfort
- Cervical arch support for greater control of flexion, extension and rotation
- Sleep mode for patient convenience
- Conforms to shoulder and mandible contours for stability and comfort
- X-ray, CT and MRI lucent

Coolbrace™ Cervical Orthosis



Product Profile

Ordering Information

Collar Height	Size	Circumference	Part Number
3-1/4"	Small	From 10" to 13"	2665-3-25-01
	Medium	From 13" to 16"	2665-3-25-02
	Large	From 16" to 19"	2665-3-25-03
	X-Large	From 19" to 24"	2665-3-25-04
4-1/4"	Small	From 10" to 13"	2665-4-25-01
	Medium	From 13" to 16"	2665-4-25-02
	Large	From 16" to 19"	2665-4-25-03
	X-Large	From 19" to 24"	2665-4-25-04

Materials

- Polypropylene collar body
- Cotton and polyurethane foam padding
- Removable and washable cotton lining
- Elastic and Velcro fastener
- Front and rear collar components
- Removeable and replaceable fan unit
- Control and battery case

Electrical

- Powered by three AA alkaline batteries, battery life 15 hr
- Three modes – OFF/SLEEP/ON
- Fan speed control knob
- Normal working current – not greater than 120mA
- Sleep mode working current – not greater than 40mA
- Stand-by current in OFF mode – not greater than 5µA
- Noise level at fan operated at highest speed – not greater than 80dB at 1" away
- Noise level of fan operated in sleep mode – not greater than 55dB at 1" away
- Fan motor life – 1000 hr

Indications

- Small, nondisplaced condylar fractures and skull base fractures extending into the occipital condyle
- Odontoid fractures above the synchondrosis level
- Stable hangman's fractures

Precautions

The physician or practitioner is responsible for providing wearing instructions and precautions to other healthcare practitioners or care providers involved in the patient's care and the patient.

If you experience any prolonged pain, swelling, sensation changes or any unusual reactions while using this product, contact VQ OrthoCare's Patient Care department at 800.452.7993 or consult your physician.

Follow the instructions of your physician for properly licensed practitioner for length and duration of use.



18011 Mitchell South, Irvine, CA 92614
 800.266.6969 Fax 800.821.8012
 www.vqorthocare.com
 customerservice@vqorthocare.com

CAUTION

Applicable federal, state, and/or local regulations may restrict this device to sale by, or on the order of, a practitioner licensed by the law of the State in which he/she practices to use or order the use of this device.