

Playmaker®

Drytex® Playmaker®

Suitable for mild to moderate ACL and/or PCL, MCL and LCL instabilities, the Playmaker® brace with extended hinge bars is an ideal intermediate-level product that can be used in sporting activities. For warm weather climates or patients that have neoprene-based irritations or allergies, The Drytex® Playmaker and Drytex Wraparound Playmaker feature a unique nylon core and polyester Lycra® fabric that allows for improved air flow.



Drytex® Wraparound Playmaker®



Through an exclusive arrangement with dj Orthopedics, VQ OrthoCare™ offers the **VQ BY DJ** brand of orthopedic products. These products add the value of VQ OrthoCare's exclusive sales, service and patient care network to the DonJoy™ line of quality products.





Product Profile



Shown w/popliteal cutout

Drytex® Playmaker®

- Polycentric hinges with contoured condyle shells
- Extension stops: available upon request zero, 10, 20, 30 and 40 degrees (10 degree installed)
- Flexion stops: 45, 60, 75 and 90 degrees (available upon request)

See sizing chart below to fill in "x"
3-0558-x w/popliteal cutout

Drytex® Wraparound Playmaker®

- Polycentric hinges with contoured condyle shells
- Extension stops: available upon request 10, 20, 30 and 40 degrees (10 degree installed)
- Flexion stops: 45, 60, 75 and 90 degrees
- Alternative to neoprene
- Lightweight, breathable fabric

See sizing chart below to fill in "x"
3-0759-x w/popliteal cutout

Sizing Chart

Measurements taken 6" above mid-patella

x=1	XS	13" - 15 1/2" (33 - 39 cm)
x=2	S	15 1/2" - 18 1/2" (39 - 47 cm)
x=3	M	18 1/2" - 21" (47 - 53 cm)
x=4	L	21" - 23 1/2" (53 - 60 cm)
x=5	XL	23 1/2" - 26 1/2" (60 - 67 cm)
x=6	XXL	26 1/2" - 29 1/2" (67 - 75 cm)
x=7	XXXL	29 1/2" - 32" (75 - 82.5 cm)

Indications

Mild to moderate ACL and/or PCL, MCL and LCL instabilities

Warnings

- If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, contact Patient Care at VQ OrthoCare, 800.452.7993 or consult a physician.
- Neoprene supports should not be worn by individuals with known susceptibility to dermatitis or allergies to neoprene byproducts.

Notice

While every effort has been made in state-of-the-art techniques to obtain the maximum compatibility of function, strength, durability and comfort, there is no guarantee that injury will be prevented through the use of this product.

CAUTION

Applicable federal, state, and/or local regulations may restrict this device to sale by or on the order of a practitioner licensed by law the of the State in which he/she practices to use or order the use of this device.



18011 Mitchell South, Irvine, CA 92614
Call 800.266.6969 Fax 800.821.8012
www.vqorthocare.com
customerservice@vqorthocare.com