

ArcticFlow™ Cold Therapy System



3-1314-0

Gravity-Fed Cold Therapy

The ArcticFlow™ cold therapy system is a gravity-fed icing device. In addition to providing effective cold therapy to the area, the specially designed elastic wraps also apply compression. The combination of ice and compression will dramatically reduce pain and swelling and help facilitate rehabilitation. Once the wrap is filled, the patient can disconnect from the unit and will be free to move about. The wrap will stay cold for up to 30 minutes, then the patient just reconnects the wrap and refilters cold water back in for up to three hours of continuous treatment.

ArcticFlow™ Cold Therapy System

Gravity-Fed Cold Therapy

Product Profile



3-1318-0
Foot/Ankle Wrap



3-1317-0
Elbow Wrap



3-1316-0
Knee Wrap



3-1315-0
Shoulder Wrap

Features and Benefits

- Lightweight one-gallon cooler – provides three hours of continuous cold
- Special wraps apply cold and compression for 30 minutes without having to recool
- Wraps available for shoulder, elbow, knee, and foot/ankle
- Quick disconnect – easy for patient to move about during treatment
- Latex-free
- Hook and loop strapping – easy application
- Gravity driven – no electricity needed, very durable
- Low cost

Indications

- Inflammation
- Edema
- Cryokinetics
- Pain
- Muscle Spasticity

Contraindications

- Cold hypersensitivity or intolerance
- Cryoglobulinemia
- Paroxysmal cold cryoglobulinuria
- Raynaud's disease
- Over regenerating peripheral nerve
- Over an area with circulatory compromise

Adverse Reactions

If used improperly, may cause tissue death, frostbite, nerve damage or unwanted vasodilation.

Ordering Information

Part Number	
3-1314-0	ArcticFlow Cooler
3-1315-0	Shoulder Wrap
3-1316-0	Knee Wrap
3-1317-0	Elbow Wrap
3-1318-0	Foot/Ankle Wrap

Warnings

Extreme care must be taken when using any cryotherapy product. Any cold product may cause frostbite if improperly used. Limit compression to avoid restricting circulation. This product is to be used under the supervision of a medical professional. This device is not intended for unsupervised public use. If you experience any pain, swelling, sensation changes or any unusual reactions while using this product, contact Patient Care at VQ OrthoCare™, 1.800.452.7993 or consult a physician.

Precautions

- Over superficial branch of a nerve
- Over an open wound
- Hypertension
- Patients with poor sensation or mentation
- Very young or very old patients



18011 Mitchell South, Irvine, CA 92614
800.266.6969 Fax 800.821.8012
www.vqorthocare.com
customerservice@vqorthocare.com

CAUTION

Federal law restricts this device to sale by or on the order of a practitioner licensed by the law of the State in which he/she practices to use or order the use of the device.

ArcticFlow is a trademark of dj Orthopedics LLC.
VQ OrthoCare is a service mark of VisionQuest Industries, Inc. © 2005