



E209137

Inflatable Back Support

Adjustable bladder provides custom fit and focused support in the lumbar area. Available with hot and cold therapy Gel Pad. Our Gel Pad helps defeat the pain cycle by providing cold therapy to reduce swelling and/or hot therapy to increase blood flow and stimulate recovery.

Airform

Inflatable Back Support

Product Profile

Features and Benefits

- Adjustable air bladder
 - Provides a custom fit and offers focused support in the lumbar area
- Self-contained pneumatic system
 - Designed to be low profile and easy to use
- Mesh panels and ventilation hole
 - These features increase breathability and enhance comfort
- Overlapping double pull strap design
 - These straps provide additional support to the lumbar and abdominal areas
- Flexible stays
 - Positioned to provide optimum support in the lumbar area
- Gripper strips
 - Rubber material is strategically sewn inside each support to prevent migration



Ordering Information

Part Number	Description
E209133	Airform Inflatable Back Support, with hot/cold therapy gel pad, Small (Waist 25" - 30")
E209135	Airform Inflatable Back Support, with hot/cold therapy gel pad, Medium (Waist 30" - 34")
E209137	Airform Inflatable Back Support, with hot/cold therapy gel pad, Large (Waist 34" - 38")
E209138	Airform Inflatable Back Support, with hot/cold therapy gel pad, X-Large (Waist 38" - 42")
E209139	Airform Inflatable Back Support, with hot/cold therapy gel pad, XX-Large (Waist 42" - 56")

Indications

- Mild to moderate low back support
- Focused compression in lumbar area
- Pain reduction
- Reduced swelling
- Increased blood flow
- Stimulates recovery

Contraindications

Cold and/or heat therapy should not be used by patients with Raynaud's disease, poor peripheral circulation, diabetes, decreased skin sensitivity or hypersensitivity to cold.

Warnings

- Extreme care must be taken when using any cryotherapy product. Any cold product may cause frostbite if improperly used.
- Limit compression to avoid restricting circulation.
- Heat therapy may cause serious burns. Do not use over insensitive skin or in the presence of poor circulation. Persons with diabetes should be especially careful applying heat.
- Children and those who are incapacitated should be supervised in their use of the hot or cold gel packs.
- If you experience any pain, swelling, sensation changes or any unusual reactions while using this product, contact Patient Care at VQ OrthoCareSM, 1-800-452-7993 or consult a physician.

Precautions

- Over superficial branch of a nerve
- Over an open wound
- Hypertension
- Patients with poor sensation or mentation
- Very young or very old patients
- Pregnancy
- Areas with poor circulation or poor thermal regulation
- Edema
- Cardiac insufficiency
- Metal
- Over a topical counterirritant (e.g. FlexAll)

Adverse Reactions

If used improperly, may cause tissue death, frostbite, nerve damage, unwanted vasodilation, burns, bleeding or fainting.

CAUTION

Applicable federal, state, and/or local regulations may restrict this device to sale by or on the order of a practitioner licensed by law of the state in which he/she practices to use or order the use of this device.



18011 Mitchell South, Irvine, CA 92614
800.266.6969 Fax 800.821.8012
www.vqorthocare.com
customerservice@vqorthocare.com

Protected by U.S. Patents 4,913,755; 4,993,409; 5,026,339; 5,062,414; 5,334,135; Re. 35,113. Other U.S. and foreign patents pending.
VQ OrthoCare is a service mark of VisionQuest Industries, Inc. © 2005

VQO20051202