

START TRACKING

Your BIO-2000 device works by keeping an accurate daily record of your progress. During the course of your treatment, your device will prompt you to answer questions relative to your progress and your responses can be tracked through the Bionicare Patient Portal website. The Patient Portal also provides useful information to help you manage your treatment and improve your condition.

- Reports for your physician
- Personalized progress graphs
- Tips for healthy living
- News stories
- Information about your device



GETTING STARTED

- 1 Insert the Bluetooth dongle into a USB port on your computer.



*Shapes of Bionicare dongle may vary

- 2 Log onto <http://www.bionicare.com/patient> and register as a user by following the instructions on the screen.



PATIENT REGISTRATION

- 3 Once registration is complete, click the green download button to install the Bionicare program by following the on-screen instructions.




After installing the program, you are now ready to track your progress and access all of the information on the BioniCare Patient Portal website!

! You must have usage on your device before uploading to the BioniCare Patient Portal website.

To check if your BIO-2000 device has connected to your computer you must turn your device off, then on again.

1) Press the BioniCare button. **2)** Press the Bluetooth button. **3)** Upon completion of upload, status will read **Complete**.

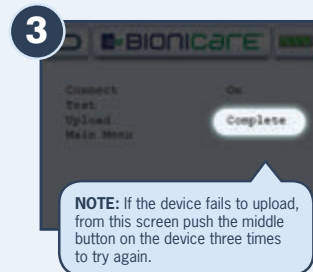

For questions, please call 800.444.1456



18011 Mitchell South • Irvine, CA • 92614
www.vqorthocare.com • 800.266.6969



© 2013 VisionQuest Industries, Inc.
VQ0511654REVE