BIONICARE SYSTEM REMOVAL STEPS

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ENGLISH

STEP 1. Unhook knee wrap from straps and carefully peel from knee. Place clear plastic backing onto knee electrode for protection when not in use.

STEP 2. Remove brace. Place clear plastic backing onto thigh electrode.

NOTE: There is no need to remove the electrodes from the Night-Wrap upon removal. Leave the electrodes in place and cover the electrodes with the clear backing to keep the electrodes clean and to prevent them from prematurely drying.

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ENGLISH

CAUTION: Electrodes should be discarded and replaced if damaged or when proper adhesive tack can no longer be obtained, the gel has separated, or you sense a change in stimulation intensity. If in doubt about the integrity or proper function of any electrode, replace it before proceeding with treatment. **If using pain medication and/or other pain management devices, exercise caution when increasing amplitude on the BioniCare device.**

WARNINGS: This device is designed for use only with electrodes cleared for medical electrical stimulation.

CAUTION: Federal law restricts this device to sale by or on the order of a practitioner licensed by the law of the State in which he/she practices to use or order the use of the device.

WARRANTY: VQ OrthoCare warranties all of its products from the original date of purchase against defects in materials and workmanship. Normal wear and tear during use of a product is not considered a defect. Contact VQ OrthoCare at 949.261.3000 for specific product warranty information.

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CARE AND MAINTENANCE OF ELECTRODES AND NIGHT-WRAP

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ENGLISH

- Do not use alcohol to clean skin surface.
- Electrodes should only be used on dry, clean, unbroken skin surfaces.
- Prior to Night-Wrap application wash skin with soap and water and completely dry.
- Do not apply skin lotion prior to application of Night-Wrap or knee wrap.
- Do not trim electrodes, as cut edges may affect the even distribution of stimulation.
- Remove Night-Wrap or electrodes before showering.
- If you experience symptoms of skin irritation while using the BioniCare system, discontinue use immediately and contact VQ OrthoCare's Patient Care Department at 800.444.1, or consult your healthcare practitioner.
- To clean Night-Wrap and liners, the electrodes should first be removed. The Night-Wrap and liners should be hand washed in cold water with mild detergent. Rinse thoroughly and air dry (do not machine dry). Never machine wash and/or use bleach, fabric softener, or harsh detergents.

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




Instructions for Use (Modo de Empleo / Mode d'emploi)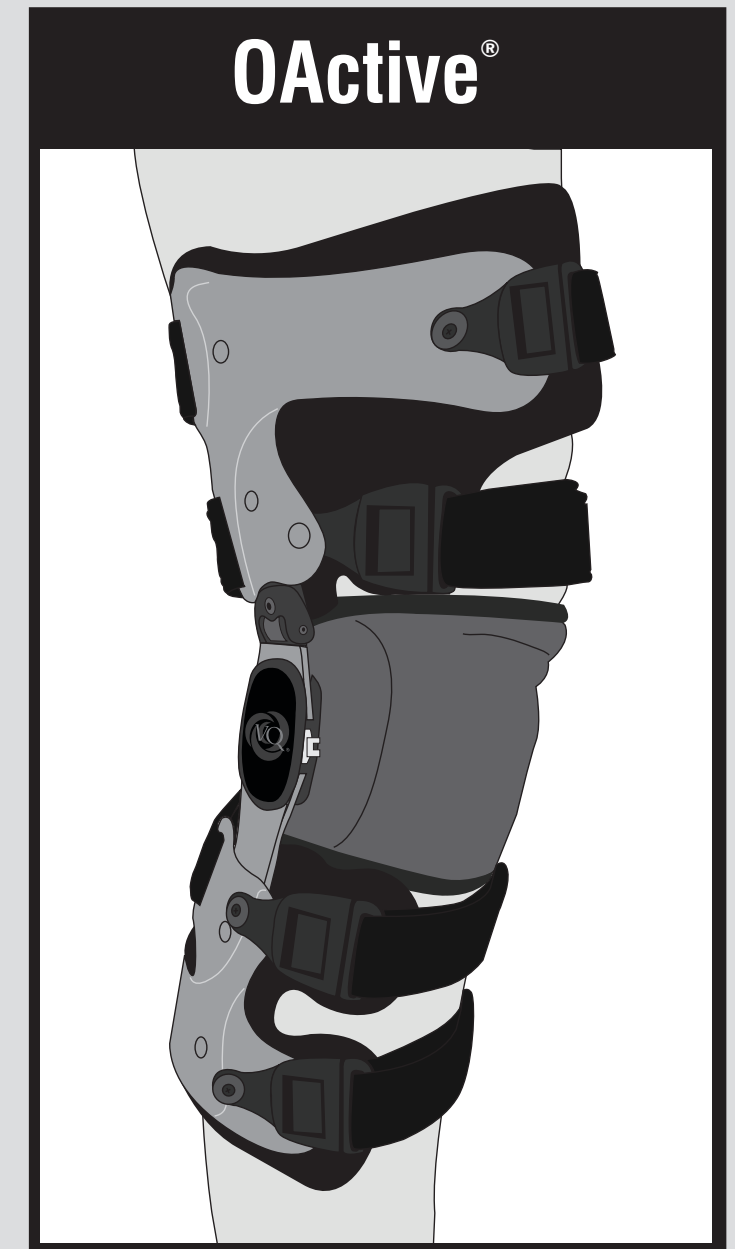

BIONICARE OACTIVE APPLICATION GUIDE

(Manual de Aplicación del Sistema BioniCare OActive / Guide de Mise en Place de L'orthèse OActive BioniCare)

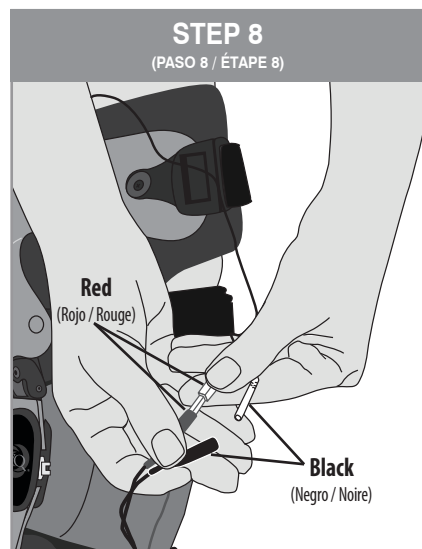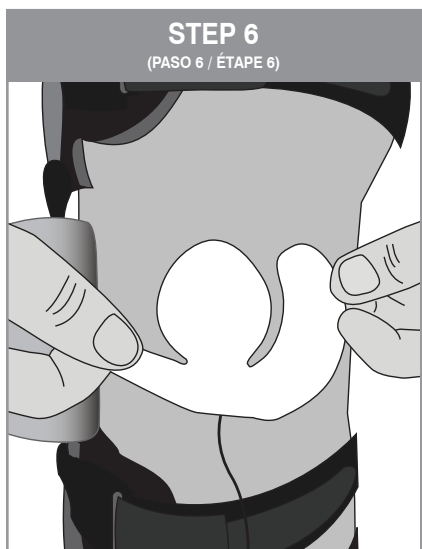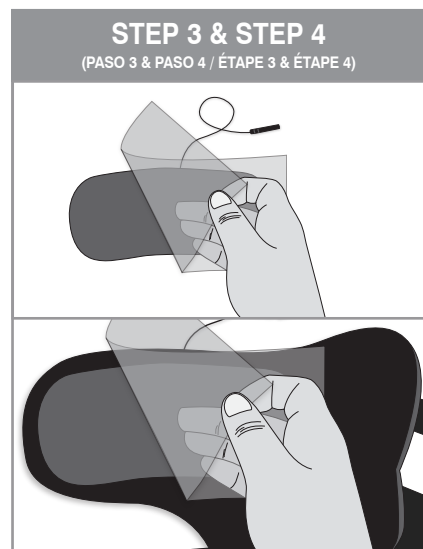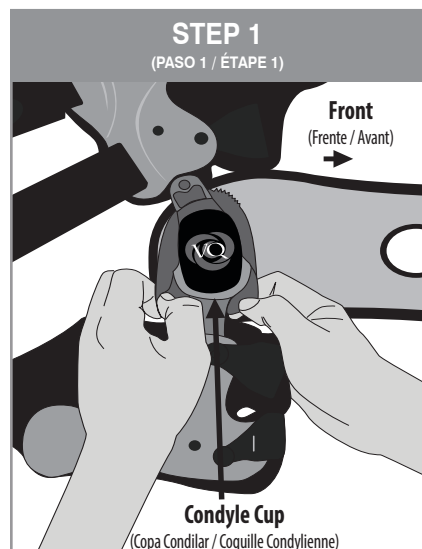ENGLISH

STEP 1. Attach knee wrap to brace. With the wrap extending out toward the front of the brace, place the condyle cup/pad into the slit in the wrap. Make sure the condyle cup/pad is completely covered by the wrap.

STEP 2. Replace thigh liner by removing the standard OActive liner from the brace and replacing it with the one included in the BioniCare kit.

NOTE: These steps may have been done for you by a technician.

STEP 3. Apply thigh electrode to thigh liner. Peel the white backing from the thigh electrode. Adhere electrode to the thigh liner, centering it in the outlined area. Make sure that the wire extends toward the top of the liner. The white side of the electrode should be against brace or liner and black side goes against the skin.*

STEP 4. Peel clear plastic gel backing. Keep plastic backing for protection of electrode when not in use.

STEP 5. Apply Brace to leg, as detailed in brace application instructions. Leave knee wrap Velcro tabs unattached.

STEP 6. Apply knee electrode. Peel clear plastic backing from knee electrode. With knee at 45°, adhere electrode to knee by first placing center section over kneecap with wire extending down as shown. Center the round area in the middle of the electrode over the kneecap. Then press entire electrode firmly to skin.

Peel white backing off electrode. The white side of the electrode should be against brace or liner and black side goes against the skin.*

STEP 7. Apply knee wrap by stretching it snugly around the knee and attach Velcro tabs on the center of the thigh and calf straps, as shown. Make sure to route wire upward towards thigh.

STEP 8. Connect lead wires. Attach red (knee) electrode wire to the red lead wire tip and the black (thigh) electrode wire to the black lead wire tip. Plug the end of the lead wire into on the signal generator.

STEP 9. Turn the signal generator on and unlock. Press the play button, then press the voltage increase button until you feel a tingling sensation. Then press the voltage decrease button until you no longer feel tingling. Recommended treatment time is a minimum of 6-10 hours, but you may increase time to get faster results.

*For demonstration purposes, electrodes are shown here in a contrasting color.

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