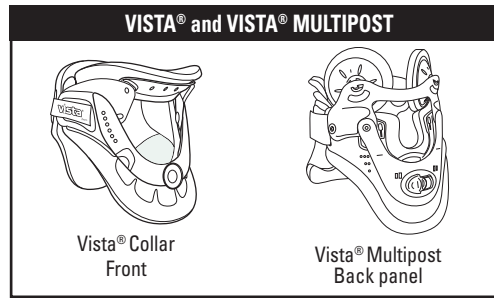
vistamp

MULTIPOST COLLAR

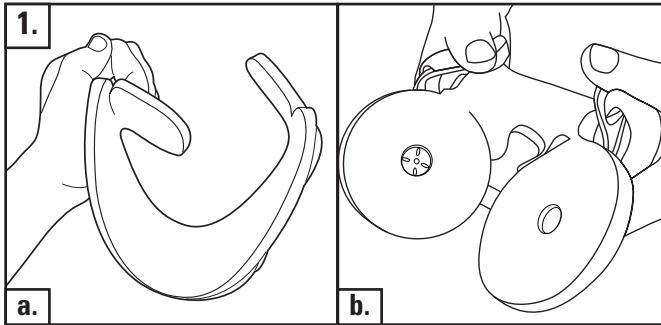
Aspen Medical Products

6481 Oak Canyon, Irvine, CA 92618-5202, USA

800-295-2776 • 949-681-0200

aspenmp.com

ENGLISH
PREPARATION

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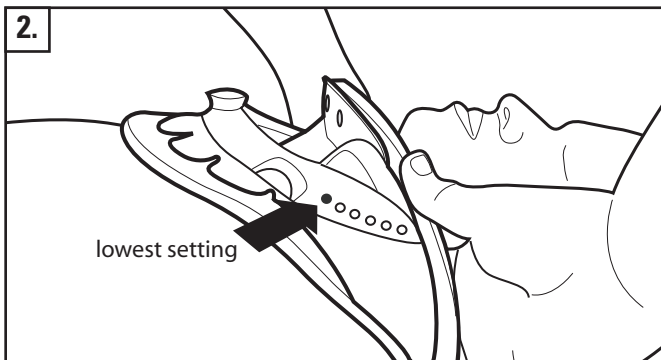
ENGLISH
PLACEMENT

The front panel should be at the lowest setting to begin. While pulling the sides of the front panel apart, position the bottom of the tracheal aperture at the sternal notch. Position the sides up and over the trapezius muscles. It is OK if the chin piece is not touching the chin.

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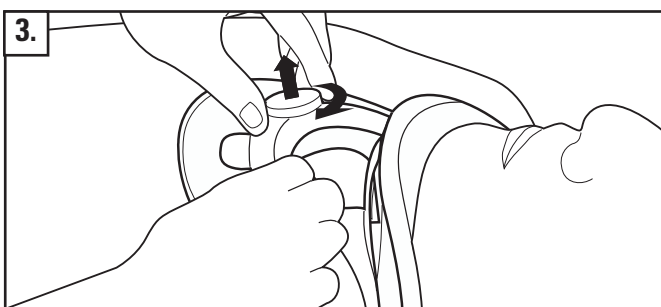
ENGLISH
SIZING

While holding the collar against the chest, pull out the dial to unlock. Turn the dial clockwise to raise chin piece. When the chin piece is supporting the chin, release the dial and it will self-lock.

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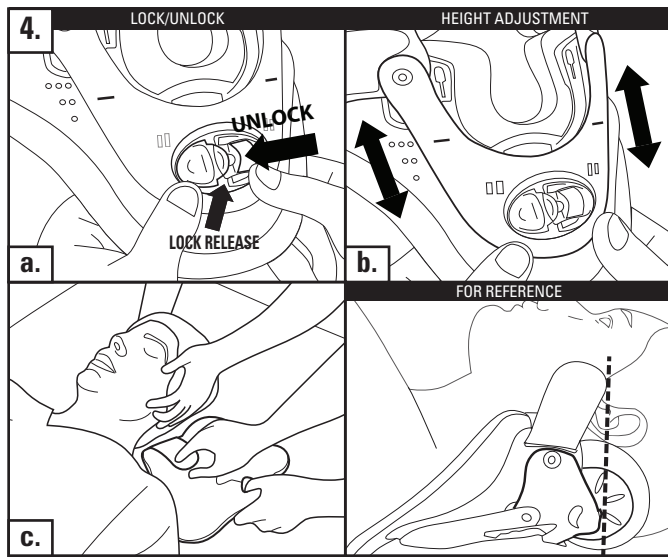
ENGLISH
MULTIPOST BACK PANEL

Note: Do not adjust on the patient. If height adjustment is necessary - remove from patient, adjust and reapply.

a. Push down on lock release and slide yellow button left to unlock.

b. Adjust height up or down if needed for best fit. To lock slide yellow button right.

c. With the head in neutral alignment, press the back panel down onto the bed and slide it behind the patient's neck. Ensure back panel has good contact with trapezius.

For Reference: Start by aligning top of the back panel with the middle of the ear.

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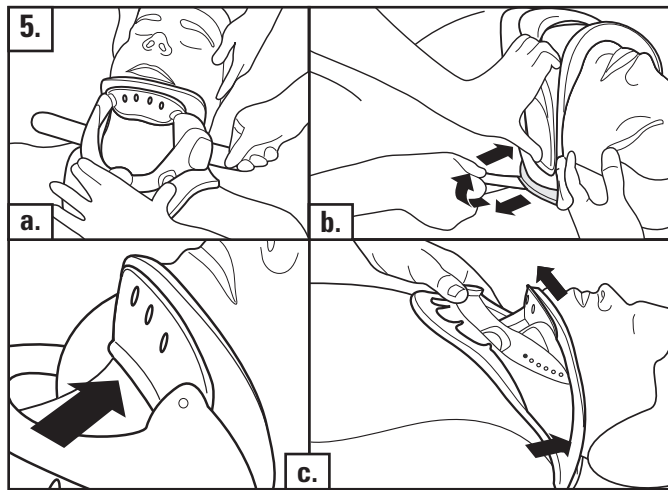
ENGLISH
TIGHTENING

a. While holding the front panel in place, attach the loop strap on each side.

b. To tighten, anchor your fingers in on the collar and peel back the loop strap on one side, tighten and reattach. Tighten the other side equally and repeat as needed to ensure a snug, symmetrical fit.

c. Check to see that the back of the chin piece is not pressing inward on the throat. If it is, lift the back ends of the side panels up and off the trapezius muscles.

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ENGLISH
TIPS

- The patient's chin should be close to the front of the chin piece but the back of the chin piece should not press on the patient's throat.
- All slack between the front and back panels must be removed to ensure a secure fit.
- The patient often changes position, so it may be desirable to readjust the height and the back panel straps.

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LIMITED WARRANTY
Aspen Medical Products, Irvine, CA 92618, warrants to the user who originally purchases this product that it is free from defects in material and workmanship. The sole obligation of Aspen Medical Products in the event of breach of warranty shall be to repair or replace the defective product or parts(s).
Aspen Medical Products shall have no obligation under this limited warranty in the event:
(a) The product was not purchased from Aspen Medical Products or through its authorized channels of distribution;
(b) The product is altered;
(c) Any parts not supplied by Aspen Medical Products are inserted into the product; or
(d) The product is not used in accordance with the Aspen Medical Products, Instructions for Use.

THE FOREGOING IS THE SOLE AND EXCLUSIVE REMEDY FOR FAILURE IN SERVICE OF, OR DEFECT IN, THE PRODUCT. ASPEN MEDICAL PRODUCTS SHALL NOT BE LIABLE UNDER THIS OR ANY IMPLIED WARRANTY FOR ANY DIRECT SPECIAL, INCIDENTAL, OR CONSEQUENTIAL DAMAGES. THIS WARRANTY IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED, INCLUDING THE WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE OR USE, AND ALL OBLIGATIONS OR LIABILITIES ON THE PART OF ASPEN MEDICAL PRODUCTS FOR DAMAGES ARISING OUT OF OR IN CONNECTION WITH THE USE OF THE PRODUCTS, WHICH ARE HEREBY DISCLAIMED AND EXCLUDED BY ASPEN MEDICAL PRODUCTS.
This warranty gives you specific legal rights and you may have other rights which vary from state to state.
Warning: To ensure proper use, please review all material.
A listing of authorized distributors in your area is available upon request.

READ INSTRUCTIONS BEFORE USE. Proper training in the use of this device should take place before it is applied. These directions are guidelines only and are not offered as medical recommendations.

THIS PRODUCT IS INTENDED FOR APPLICATION BY LICENSED HEALTH CARE PRACTITIONERS AS DIRECTED BY A PHYSICIAN OR OTHER QUALIFIED MEDICAL AUTHORITY. PERMANENT INJURY MAY RESULT FROM USE OTHER THAN AS DIRECTED BY A PHYSICIAN OR OTHER MEDICAL AUTHORITY. ASPEN MEDICAL PRODUCTS SPECIFICALLY DISCLAIMS ANY OBLIGATION OR LIABILITY FOR INJURY OR DAMAGE BY REASON OF ANY APPLICATION OF THIS PRODUCT. PLEASE SEE THE ACCOMPANYING LIMITED WARRANTY FOR FURTHER INFORMATION IN THIS REGARD.

PATENT PENDING