ENGLISH

RANGE OF MOTION ADJUSTMENTS:

The Catalyst•Propel OA comes with standard 5 degree extension stops. Additional flexion and extension stops may be installed if they are prescribed by your physician or needed for optimal performance or fit of the brace. These should be inserted by the technician who fits your brace. Optional 0°, 10°, 20°, 30°, and 40° extension stops and optional 45°, 60°, 75° and 90° flexion stops are included with each brace.

IMPORTANT: Both hinges must be set to same extension and/or flexion settings. For 0° extension limiting, 0° extension stops must be used.

TO CHANGE AN EXTENSION STOP:

Unscrew the screw on the front of the hinge cap. Shake out the 5 degree extension stop. Insert another extension stop into the front of the hinge and replace the screw. Repeat for the opposite hinge.

TO INSERT A FLEXION STOP:

Insert a flexion stop into the back of the hinge and use the extra screw that is attached to the flexion/extension stop tree to secure. Repeat for the opposite hinge.

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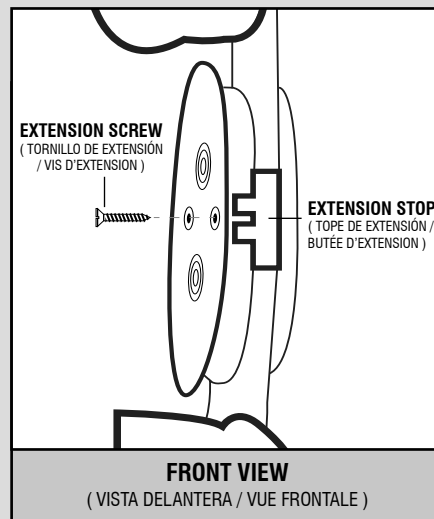
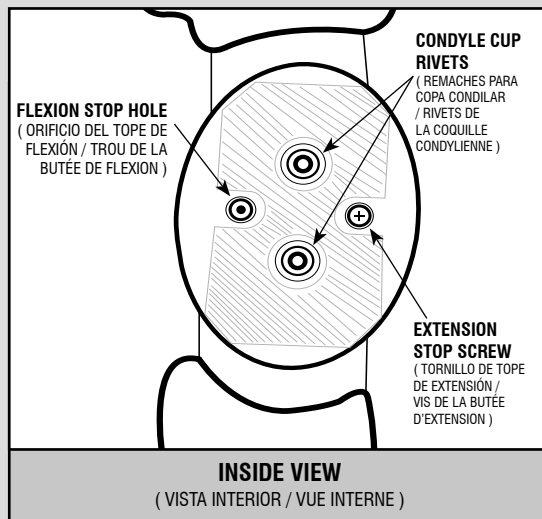
ENGLISH

INDICATIONS:

- Mild to severe unicompartmental knee osteoarthritis with ligament istabilities
- Increased rehabilitative and functional support for diagnoses that require reduced medial or lateral joint loads

WARNINGS:

If you experience pain, swelling, sensation changes, or unusual reactions while using this product, contact VQ OrthoCare's Patient Care department at **800.452.7993** or consult a physician.

FOR SINGLE PATIENT USE ONLY.

NOTICE: While every effort has been made using state-of-the-art techniques to obtain maximum compatibility of function, strength, durability and comfort, this device is only one element in the overall treatment program administered by a medical professional. There is no guarantee that injury will be prevented through use of this device.

CAUTION: Applicable federal, state, and/or local regulations may restrict this device to sale by or on the order of a practitioner licensed by law of the State in which he/she practices to use or order the use of this device.

WARRANTY: VQ OrthoCare warranties all its products from the original date of purchase against defects in materials and workmanship. Normal wear and tear during use of a product is not considered a defect. Contact VQ OrthoCare at **800.266.6969** or **949.261.3000** for specific product warranty information.

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EMERGO EUROPE
Molenstraat 15
2513 BH, The Hague
The Netherlands

Subject to one or more patents listed on
www.vqorthocare.com/patents

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MANUFACTURED AT
1390 Decision St, Suite A
Vista, CA 92081, USA

© 2015 VisionQuest Industries Inc.
VQO151773REVC

Instructions for Use

(Modo de Empleo / Mode d'emploi)

CATALYST•PROPEL™ OA

Osteoarthritis Knee Brace

(Rodillera Para Osteoartritis / L'arthrose Orthèse de Genou)

Catalyst•Propel OA (Active Thigh Cuff)

Catalyst•Propel OA (Manguito Activo Para El Muslo) /
Catalyst•Propel OA (bande cuisse "ATC")

CATALYST•PROPEL OA APPLICATION INSTRUCTIONS

(Instrucciones De Aplicación De Catalyst•Propel OA / Instructions d'application Catalyst•Propel OA)

STEP 1.

(PASO 1. / ÉTAPE 1.)

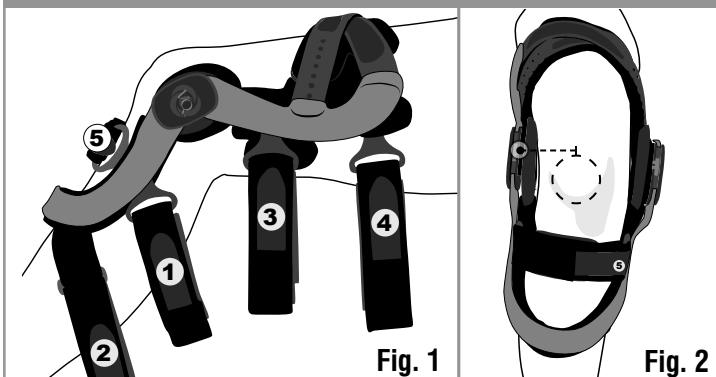
Fig. 1

Fig. 2

STEP 2.

(PASO 2. / ÉTAPE 2.)



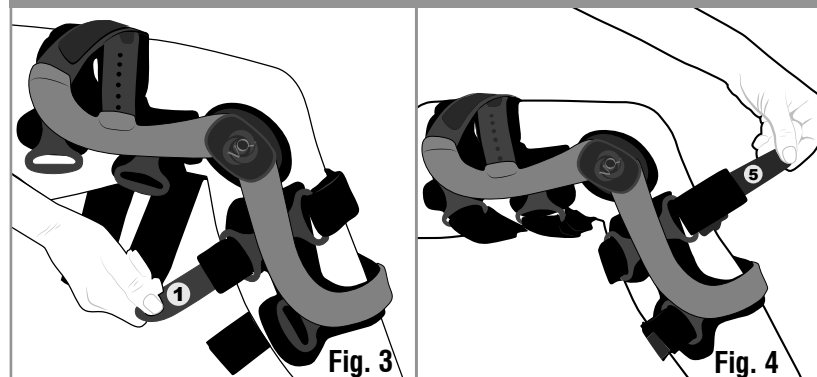
Fig. 3

Fig. 4

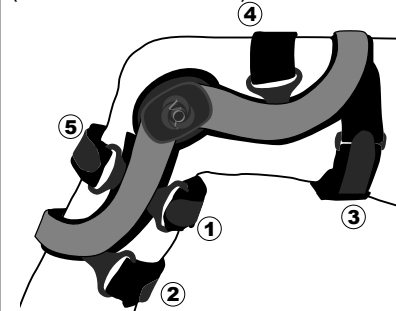PCL VERSION (VERSIÓN LCP / VERSION LCP)


CI VERSION (VERSIÓN ILC / VERSION LC)

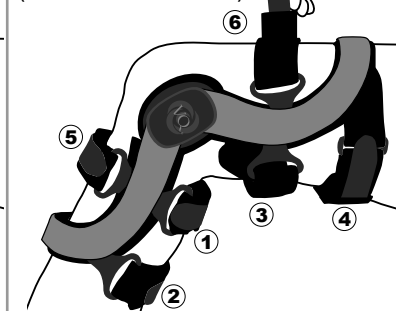ATC MODELS

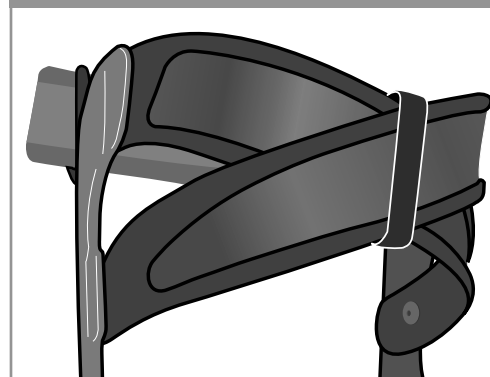(MODELOS CON ATC / MODÈLES ACTIVE THIGH CUFF)

STEP 3.

(PASO 3. / ÉTAPE 3.)

ENGLISH

The VQ OrthoCare Catalyst•Propel OA brace is designed to reduce the pain of your knee osteoarthritis and provide stability.

The Catalyst•Propel OA also features the unique Active Thigh Cuff™ (ATC), a patented thigh-strapping mechanism that dynamically adapts to the shape and contour of the thigh throughout a full range of motion. The ATC provides optimal comfort and brace suspension, reduces fatigue, and minimizes brace migration during activity.

OA CORRECTION

Your brace comes with a thin and thick condyle pad. Place the thin condyle pad on the affected side of your knee. Place the thick pad on the opposite side so that it provides correction. For custom braces, these have been placed correctly in the brace for you during manufacturing.

STEP 1.

Unfasten all four posterior (behind leg) straps and fold strap ends back onto themselves to prevent tangling. In a seated position with leg slightly bent, place brace on leg so that center of hinge is lined up with top of knee cap (see figs. 1 and 2). Hinge center is marked by the gray extension stop.

STEP 2.

Attach straps in sequence indicated on strap ends. Hold brace at hinge to prevent movement while tightening straps.

Strap #1: Fasten upper calf strap and position above calf muscle (see fig. 3).

Strap #2: Fasten lower calf strap.

Strap #3: Fasten lower thigh strap.

Strap #4: Fasten upper thigh strap securely but not too tightly. Overtightening may restrict circulation and/or result in downward migration of brace.

Strap #5: Fasten anterior (front) strap and tighten securely (see fig. 4).

NOTE: From side view, hinges should sit slightly posterior (behind) to midline of knee. Adjustment of straps #1, #3, and #5 will influence this alignment.

STEP 3.

Stand up and take a few steps. Brace will drop slightly to align with center of kneecap. If adjustment is needed, hold brace to prevent rotation while straps are repositioned.

ATC ADJUSTMENT

Catalyst•Propel braces with the Active Thigh Cuff (ATC) may be adjusted further if the uprights are not positioned along the midline of the thigh or if there is gapping between the soft tissue and the ATC straps. ATC adjustment should be done by your healthcare professional or brace technician. Remove the screws for each strap. Adjust as needed to fit flush against the thigh with the side flanges sitting mid-medial and mid-lateral or slightly posterior. Replace screws. Trim as needed.

ATC MODELS

If desired, install ATC pivot clip where the ATC components cross, as shown. The pivot clip allows the ATC to pivot in the center during use and is there as an option if the crossing straps are moving too much or shift out of their proper place on the leg.

Also included are rigid ATC straps that can be used in place of the flexible straps already installed if additional rigidity is desired.

TROUBLESHOOTING

It is normal for the brace to drop slightly after initial application so the hinge centers align with the center of the kneecap. If the brace drops below knee center, reapply the brace.

Excess soft tissue, undefined calf muscles or unusual leg contour may require use of an undersleeve to help suspend your brace. Contact your orthotic professional for more information about undersleeves.

If you have questions regarding the fit of your brace, contact VQ OrthoCare's Patient Care Department at **800.452.7993** or your local brace provider.

CARE AND MAINTENANCE

- Salt water, sand, dirt and other elements can cause damage. Thoroughly rinse brace with fresh water after exposure to these conditions. Towel or air dry.
- Liners should be removed and hand washed as needed using cold water and mild detergent. Rinse thoroughly and air dry (Do not machine wash. Do not machine dry).
- Lubricate hinge with dry silicone spray as needed.

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